

# Texas Catheter®

Disposable External Male Catheter

1 DOZEN

WITH ELASTIC FOAM STRAPS

REORDER NO. 8884-

731300

FOR MALE URINARY INCONTINENCE



Soft, lightweight latex sheath provides snug fit without constriction.

Connector permits attachment to urinary leg bag or other collection receptacle.

812791

Elastic foam strap comfortably holds catheter in place and prevents backflow. Accommodates to all sizes and conditions.



## DIRECTIONS FOR USE

# Texas Catheter<sup>®</sup> Disposable External Male Catheter

1. Position TEXAS Catheter<sup>®</sup> device ½ inch in front of head of penis.
2. Roll latex sheath over penis so that enough sheath extends behind head of penis to allow attachment of tape.
3. Secure catheter. If using product which includes elastic foam tape, peel backing from tape and, with adhesive side down, spiral tape snugly around sheath, beginning behind head of penis.
4. Attach connector on end of sheath to leg bag or to drainage tubing leading to leg bag or other receptacle.

This is one of the suggested techniques for securing the device. Anatomical considerations may require alternative methods of attachment.

**IMPORTANT:** To avoid impairing circulation, tape should be snug but not too constrictive. Check 15 minutes after attaching tape to be certain there is no swelling. (Check more often if patient's condition dictates.)

**IMPORTANT:** We urge you to follow instructions provided for the proper use and storage of this product. This product contains natural latex. Do not store near heat or keep exposed to light.

ASSEMBLED IN MEXICO



039-0390  
086018